SUPERFIX 25 RTU

Application

Specially designed for table processors and processors with small tank volumes. **SUPERFIX 25 RTU** is suitable for all areas of X-ray diagnostics, including dental film processing and fixing X-ray films for non-destructive testing (NDT).

Characteristics

SUPERFIX 25 RTU has a high fixing speed, and so ensures complete film fixing during the short treatment period, even in fast-running processors. Most screenless films can be processed in a cycle of just 3.5 minutes. The fixer will keep for at least 12 months at storage temperatures between 5°C and 30°C. Freezing and excessive heat should be avoided.

Pack sizes

SUPERFIX 25 RTU is a ready-to-use solution and is supplied in a pack of 2 x 5 litres.

Fixer temperature

Should not be more than 1°C to 2°C under the developer temperature, but at least 25°C.

Replenishment

SUPERFIX 25 RTU should be replenished at 70 to 80 ml per 35 x 35 cm sheet. If the percentage of screenless films is over 5%, raise the fixer replenishment rate by 10%.

Mixing instructions

The product is ready to use.

When filling the machine tank, make sure that no fixer solution gets into the developer tank. Therefore it is advisable to put the fixer in first.

Technical data

pH of ready-to-use solution: 4.1 - 4.3 (20°C)

Specific gravity of ready-to-use solution: 1.073 - 1.083 (20°C)

Certification

TETENAL applies a quality management system certified to DIN EN ISO 9001.

This product complies with directive 93/42 EEC of 14th June 1993 with regard to medical products (CE labelling).

Tetenal AG & CO.KG · Schützenwall 31-35 · 22844 Norderstedt/Germany · Tel.: 040-521 45 0 · Fax: 040-521 45 0 · eMail: sales@tetenal.com

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